
medoVital

Natur-Moor | natural moor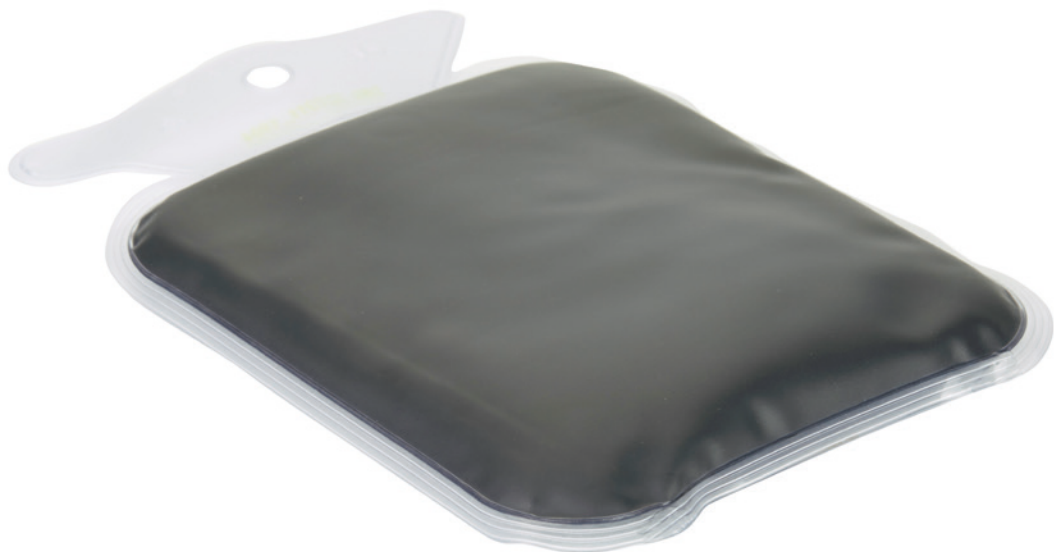

Wärmekissen
heat cushion

Nackenkissen
neck cushion

Rückenkissen
back cushion

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Instructions for use

Thank you for buying a natural moor cushion from bb med.! The product is a medical device. It provides gentle healing and relaxation with natural ingredients. The natural moor cushion releases therapeutic heat over the course of several hours.

The moor filling is a modified natural product. Colour variations and discolourations occur naturally. The peat is obtained from regenerative extraction in Germany. The film used is free from critical softeners. The natural moor cushion is produced in Germany, in the Lower Rhine region.

Please observe the following instructions. This will help you to enjoy the comfort of the high quality natural moor cushion for many years. If you have any questions, please ask your physician or pharmacists.

Application in heat therapy – ideal for tensions

Muscles relax at a temperature of 40 – 43 °C and the body's regenerative powers are strengthened. The increased circulation stimulates the metabolism, improves the natural healing process and increases well-being. Long term heat relieves pain!

Application in heat therapy: lumbago, sore muscles, back pain, stomach pain, period pain, neck pain, tenseness, rheumatic pain (please ask your physician or therapist before use; do not use for rheumatoid inflammation!), stiff joints or neck.

Article	moor heat cushion	moor neck cushion	moor back cushion
microwave (at max. 600 Watt) stop every two minutes and knead the cushion	heat for approx. 2-3 minutes, continue in 10 second intervals if required	heat for approx. 2-3 minutes, continue in 10 second intervals if required	heat for approx. 3-4 minutes, continue in 10 second intervals if required
oven place on a plate (at max. 120 °C) do not preheat	heat for approx. 10 minutes, continue in 2 minute intervals if required	heat for approx. 10 minutes, continue in 2 minute intervals if required	heat for approx. 15- 20 minutes, continue in 2 minute intervals if required

Safety information – caution: Do not leave the moor cushion unattended while heating! If the product inflates, switch off the microwave or oven and check the moor cushion for damage when it has cooled down. Overheating the cushion can result in cracks and leaks. If this is the case, do not use the cushion. Do not let the moor cushion come into contact with hot metal.

Application in cold therapy – ideal for inflammations

Cold reduces inflammations as the lower body temperature inhibits the activity of the inflammation mediators. Cold therapy initially triggers blood vessel reactions and the muscles contract. The sensation of pain is reduced. Longer application reduces muscle activity, lowers muscle tension and eases cramps.

Application in cold therapy: bruises, sprains, pulled muscles, headaches, migraines, fever, toothache, insect bites, nose bleeds, contusions, nocturnal leg cramps, oedema, tennis elbow, inflammation of muscles and joints, reviving tired and heavy legs, rheumatic pains (please ask your physician or therapist before use).

Article	moor heat cushion	moor neck cushion	moor back cushion
refrigerator	permanent storage possible	permanent storage possible	permanent storage possible

Important information

Ensure to carefully check the temperature with the back of your hand before use. To avoid possible heat or cold reactions on the skin, always place the cushion into a towel or a protective cover after activation. Please remember that burns can occur at temperatures as low as 48 °C. Keep children away from packaging materials! Risk of e.g. suffocation. Do not use the moor cushion on persons who are unable to feel heat or cold. Therefore please do not use this article on children under 3 years, helpless persons or heat-insensitive persons. Do not use in combination with painkillers. Do not apply cold therapy if you suffer from arterial circulatory problems and/or Raynaud syndrome.

Do not use the cushion if there is visible damage which may be caused e.g. by sharp or pointed objects, overheating or excessive impact (e.g. dropping the cushion on the floor)! Please check the cushion for damage before each application, after the heating process. If the moor cushion leaks, do not use it as other objects may become soiled. We can therefore not provide any compensation for possible follow-up damage.

Storage and care: We recommend storing the cushion in the refrigerator. The instructions for use have to be stored in a safe place. After use, clean the cushion with a damp cloth and leave to dry. Do not use any aggressive cleaning agents or solvents. When the moor cushion is not going to be used for a longer period of time, it should be heated periodically to prevent it from drying out.

We hope you enjoy using this product.

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MedoVital Handels GmbH
bb med. product GmbH